



How to Install IntelViewer on a Windows PC

1. Go to Bay Radiology's website <https://bayradiology.co.nz/>
2. Access the Medical Practitioners Tab > Online Image Access > IntelViewer Downloads
3. Select the appropriate **Windows** IntelViewer Installer to download
4. After downloading the software double click the executable to launch the installer.
5. Read and accept the terms of the license agreement, and then click Next
6. The Destination folder window appears. If you wish to select a different folder in which to install the application, click change and select the folder otherwise leave this as default.
7. Click Ok, and then click Next
8. Choose "All users" and click Next
9. Choose "Disable DICOM service" and click Next
10. Select "Create Shortcuts on my desktop" and Click Install
11. The below icon will appear on your desktop.

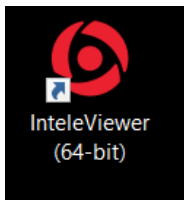

12. Double click on the application icon on the desktop, you will be prompted to enter a Server Address. Enter: <https://intelepacs.bayradiology.co.nz>
13. Tick Protect Bookmark and Use Custom Server Name
14. In the Custom Server Name add in "Bay Radiology & Medex"

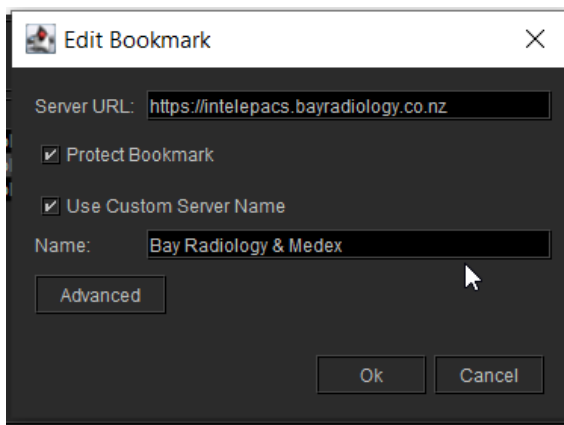

How to Install IntelViewer on a MAC

1. Go to Bay Radiology's website <https://bayradiology.co.nz/>
2. Access the Medical Practitioners Tab > Online Image Access > IntelViewer Downloads
3. Select the MAC IntelViewer installer to download
4. Locate the IntelViewer.dmg file, open the file and drag the IntelViewer application to the Applications Folder

5. Open the application folder, right-click the IntelViewer icon and click Open
6. A message appears stating that IntelViewer cannot be opened due to an unidentified developer. Click Open
15. If this is the first time accessing the application, you will be prompted to enter a Server Address. Enter: <https://intelepacs.bayradiology.co.nz>
16. Tick Protect Bookmark and Use Custom Server Name
17. In the Custom Server Name add in "Bay Radiology & Medex"
18. For quick access to IntelViewer, you can add the IntelViewer application icon to the dock. Right-click the IntelViewer icon in the dock, click options and then click "Keep in Dock"